

This square  
measures one  
inch. Ensure  
your printout  
is accurate.

**Easyboot®**  
**Easyboot® Epic**  
**Easyboot® bare**



**EasyCare**  
800.447.8836  
[www.easycareinc.com](http://www.easycareinc.com)

#pony

#000

#00

#0

#1

#2

#3

#4

#5

#6

#7

#### TO DETERMINE BOOT SIZE:

- 1) Take a separate plain piece of paper and have your horse step on it. You may also hold the hoof up and place a piece of paper over it.
- 2) With a dark pen or a marker, trace around the edge of the hoof onto the paper. Make sure that you get very close to the hoof edge.
- 3) Place your traced hoof drawing behind the sizing chart and hold them up together in front of a light source or a window.
- 4) The traced line should match closest to the dotted lines on the chart for the ideal boot size.
- 5) If you have already pre-measured your horse's hoof you may wish to trim the template down to size and hold that up to the hoof.
- 6) Make sure you measure the one inch square box on this form to ensure that the template printed to size.